

# OVERHEAD LIFT

## READY TRACK REFERENCE GUIDE

### Manual Emergency Lower

May be used in event of power failure or lift malfunction.

### Auxiliary UP/DOWN Switches

May be used in event handset does not operate.

POWER - GREEN light indicates power ON. DIAGNOSTIC - Flashes RED in various sequences indicating various internal trouble situations. Solid RED light indicates low battery condition. Battery should be charged immediately. CHARGING - YELLOW indicates active charging. GREEN indicates fully charged.

### Auxiliary POWER/ Emergency DOWN switches

For use if Handset has malfunctioned. EMERGENCY DOWN may be used to lower patient in event of malfunction in lift circuitry.

Power ON/OFF

UP/DOWN

Emergency down

Handset

Emergency OFF Pull Cord Switch

If lift does not operate and POWER LED does not light, pull cord once gently to turn power ON. May be used to cut power to lift entirely in an urgent situation.

### HELPFUL HINT

#### Return to charge after every use.

Note: some hand control models will have RTC (return to charge) button. Motor will attempt to return to docking station and if track configuration is a H-track you will need to manually push motor to charging dock.



**HUMAN CARE**  
Your Life. Your Way.


1



Select and position the sling around the patient according to the appropriate sling application instructions.

2

Using the carrybar, pull the lift to position the carrybar above the patient.

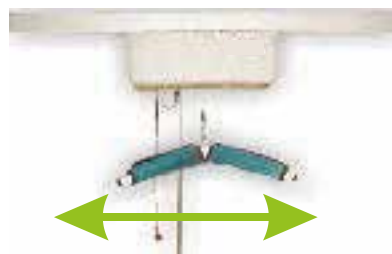

Lower the carrybar (DOWN button on handset) to the patient.




3

Attach the sling loops to the carrybar so the patient is properly positioned.



4

Raise the patient (UP button on the handset) As the carrybar raises, check to ensure the straps are firmly in place.

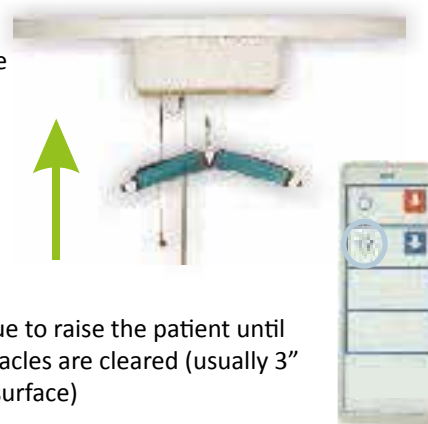

5

Gently maneuver the patient horizontally along the track to desired position.



Continue to raise the patient until all obstacles are cleared (usually 3" of the surface)

Use the DOWN button on the handset to lower the patient to the surface. Use the grab handles on the sling to help position the patient as they are lowered. Continue to lower until all straps are slack.



6

Remove the straps from the carrybar and raise the carrybar out of the way (UP button on handset). Ensuring all obstacles are cleared from its path, use the carrybar to pull the lift to its "docking" position. Ensure the Yellow "Charging" light is on (LED panel)



**Human Care USA**  
 4210 S Industrial Suite 160  
 Austin, TX 78744  
 Phone:- 512 476 7199  
 Toll free number: 877 822 7866  
 Fax:- 512 476 7190  
 Email:- info.us@humancaregroup.com

